

Hot Cold Therapy Gel Waist Belt



Item No: Y52
Material: Gel Packs + Neoprene Wrap
Size : one size suit all
Color: green blue
Packing: 1 piece / opp bag,50 pcs/carton

The gel packs contour around your midriff and sides around the waist and attaches with fastening strap closure. Can be used on the hip, across the back of the torso, the belly and more.

Common Examples of Use

Use cold compress

1. Place the product in the freezer for at least 2 hours.
2. Take it out and put it directly on the body or put it in a cloth cover and put it on the body for 20 minutes.

Use hot compress:

1. Boil a pot of hot water, soak the product for 10 minutes, take it out and use it for hot compress on the human body.
2. Put the product in the microwave oven and heat it for 20 seconds over medium-low heat. If the temperature is not satisfactory, heat it again for 10 seconds. Then, take it out and use it for hot compress on the human body.

Product Features

1. The product can be used as a cold compress or as a hot compress.
2. The product fabric is durable, can be reused, and remains soft after freezing.
3. Various shapes, sizes and colors can be customized

.Size Chart

one size suit all